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Instructions for use
Siccaforte®
(0.2 % carbomer)

Siccaforte®
Eye gel

Description
Siccaforte is a colorless gel with dropping consistency, which has very similar characteristics to the natural tear fluid of the eye. Therefore, Siccaforte is used as a replacement for the natural tear fluid to moisten and lubricate the surface of the eye. Therefore, it gives long lasting relief of dry eyes caused by:

- chronic dry eye symptoms (Sjögren 's Syndrome)
- corneal injuries, corneal erosion, corneal surgery
- hormonal imbalances (menopause)
- environmental stress induced by air conditioning, wind, cold climate, sun, dry heat, dust, smoke and air pollution
- digital eye strain from computers or TV, long car journeys or flights
- the use of certain medicines (e.g., antidepressant, antihistaminic, neuroleptic, hormones)
- swimming in chlorine containing water

Siccaforte can be used up to **6 weeks after opening**. Continue use with a new tube.

Composition:
Carbomer 0,2 %, cetrimide, sodium hydroxide, sorbitol (E420), water for injections.

Application:
Always use Siccaforte according to instructions of this leaflet. If your doctor did not tell you otherwise, follow the dosage mentioned in this leaflet. The Siccaforte should be administered in the conjunctival sac of the eye. Remove contact lenses before use. Wait 15 minutes before re inserting contact lenses.

Adults:
Administer 1-2 drops into the conjunctival sac 2-4 times daily. Hold the tube upright.

Precautions and Warnings:

- Do not use if security seal is broken (⊗).
- Do not touch the eye with the opening of the tip.
- Avoid contact with the tube opening to prevent contamination.
- Write opening date on the tube!
- Dispose the tube 6 weeks after breaking the security seal.
- Do not use if you are allergic to any if the ingredients or have local hypersensitivity.
- Do not share the product (⊕).
- If the product is used together with other eye drops or eye ointment an approx. 15 minutes interval between the applications should be kept. Siccaforte should always be used last.
- If you have any further questions or eye discomfort, contact your doctor or pharmacist.
- Please contact the manufacturer or the competent national authority in case of occurrence of serious incidents related to the product.

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